

# Gear View QC Installation and Setup Guide

LX-DOC-GVQC2.3.1-IN-EN-REVB

# **Regulations and Compliance**

## **International Sponsors**

#### **Australian Sponsor:**

Emergo Australia Level 20 Tower II Darling Park 201 Sussex Street Sydney, NSW 2000 Australia

### **EU Sponsor:**

EC REP

## Brazil Sponsor:

Gear View Software

Registro Anvisa nº: 80117589010

Responsável Técnico: Luiz Levy Cruz Martins CRF- 42415

IMPORTADOR:

Emergo Brazil Import Importação e Distribuição de Produtos Médicos Hospitalares Ltda. Avenida Francisco Matarazzo, 1.752, Salas 502/503, Água Branca, São Paulo-SP,

CEP - 05001-200 -Cidade: Brasília / UF: DF CNPJ: 04.967.408/0001-98

e-mail: brazilvigilance@emergogroup.com

 $\epsilon$ 



**Hyland LLC** 4309 Hacienda Drive, Suite 500 Pleasanton, CA 94588 USA

Tel: 1-844-535-1404

**EMERGO EUROPE** Prinsessegracht 20

2514 AP The Hague

The Netherlands

Email: TS\_PACSGEAR@hyland.com

© 2018 Hyland. Hyland and the Hyland logo are trademarks of Hyland LLC, registered in the United States and/or other countries. All other trademarks are the property of their respective owners.

# Table of Contents

Regulations and Compliance	2	
Download the Gear View QC setup program	. 4	
Install the Gear View QC service	4	
Install the Gear View QC client		
License Gear View QC		
Log in for the first time	5	

# Download the Gear View QC setup program

To download the Gear View QC setup program to the host computer, complete one of the following options.

- From a browser, go to the URL provided by your support representative. On the Download page, click the link. If prompted, specify a destination directory and click OK.
- Insert the Gear View QC installation disc into the CD-ROM drive of the host computer. In Windows, copy the installation directory from the CD-ROM to the host computer

## Install the Gear View QC service

To install the Gear View QC service, complete the following steps.

- 1. Navigate to the GEARView QC Service folder, and then double-click setup.exe.
- 2. If the Windows UAC prompt appears, click Yes.
- 3. In the Select the language dialog box, select a language in the list and then click OK.
- 4. In the Welcome to the InstallShield Wizard for GEARView QC Server dialog box, click Next.
- 5. On the License Agreement page, scroll to the bottom of the license agreement and click Yes.
- 6. On the **Select Features** page, complete one of the following options.
  - To install a new SQL database, select all features and click Next.
  - To use an existing or remote database, clear the Microsoft SQL Express 2008 check box, click Next, type database details, and then click Continue.
- 7. If no message appears, skip to the next step. If the message, The installation process has detected that a compatible version of SQL Server has already been installed appears, click OK, and then complete the following substeps.
  - 1. In the **Database server that you are installing to** list, select or browse for a database.
  - 2. Under Connect using, click an authentication method.
  - 3. In the Name of database catalog box, browse for the catalog.
  - 4. Click Next.
- 8. On the Ready to Install the Program page, click Install.

**Note:** If prompted, follow the on-screen instructions to install any missing requirements or prerequisites.

9. On the InstallShield Wizard Complete page, click Yes, I want to restart my computer now, and then click Finish.

## Install the Gear View QC client

To install the Gear View QC client, complete the following steps.

- 1. Navigate to the GEARView QC Client folder and double-click setup.exe.
- 2. If the Windows UAC prompt appears, click Yes.
- 3. In the **Select the language** dialog box, select a language in the list and click **OK**.
- 4. In the Welcome to the InstallShield Wizard for Gear View QC Client dialog box, click Next.
- 5. On the License Agreement page, scroll to the bottom of the license agreement and click Yes.
- 6. On the Ready to Install the Program page, click Install.

**Note:** If prompted, follow the on-screen instructions to install any missing requirements or prerequisites.

7. On the InstallShield Wizard Complete page, click Finish.

## License Gear View QC

To license Gear View QC, complete the following steps.

- 1. To launch the Gear View QC client program, double-click the **GEARView QC desktop** icon.
- 2. In the GEARView QC License dialog box, in the System ID box, note the system ID.
- 3. To obtain a license key, send an email to key-pacsgear@hyland.com. In the body of the email, type the site name, city, state or province, country or region, product name, system ID, and department or workstation.
- 4. When you receive your license key, enter it in the **License key** box.
- **5.** To install a demo, select the **Activate demo** check box. In the **Month**, **Date**, and **Year** boxes, type the expiration date that was included with your license key.
- 6. Click **OK** to display the **Log On** dialog box.

## Log in for the first time

After you license Gear View QC, the Log On dialog box appears. To log in, complete the following steps.

- 1. In the User name box, type admin.
- 2. Leave the **Password** box blank.
- 3. In the Server box, accept the default value of localhost or type the IP address of the PC on which you installed the Gear View QC server.
- **4.** In the **Port** box, accept the default value of 8000, type the port number of the PC on which you installed the Gear View QC server, and then click **OK**.