



# Gear View QC Release Notes

DOC-GVQC3.2-RN-EN-REVA

# Documentation Notice

Information in this document is subject to change without notice. The software described in this document is furnished only under a separate license agreement and may only be used or copied according to the terms of such agreement. It is against the law to copy the software except as specifically allowed in the license agreement. This document or accompanying materials may contain certain information which is confidential information of Hyland Software, Inc. and its affiliates, and which may be subject to the confidentiality provisions agreed to by you.

Complying with all applicable copyright laws is the responsibility of the user. Without limiting the rights under copyright law, no part of this document may be reproduced, stored in or introduced into a retrieval system, or transmitted in any form or by any means (electronic, mechanical, photocopying, recording, or otherwise), or for any purpose, without the express written permission of Hyland Software, Inc. or one of its affiliates.

Hyland, HXP, OnBase, Alfresco, Nuxeo, and product names are registered and/or unregistered trademarks of Hyland Software, Inc. and its affiliates in the United States and other countries. All other trademarks, service marks, trade names and products of other companies are the property of their respective owners.

© 2025 Hyland Software, Inc. and its affiliates.

The information in this document may contain technology as defined by the Export Administration Regulations (EAR) and could be subject to the Export Control Laws of the U.S. Government including for the EAR and trade and economic sanctions maintained by the Office of Foreign Assets Control as well as the export controls laws of your entity's local jurisdiction. Transfer of such technology by any means to a foreign person, whether in the United States or abroad, could require export licensing or other approval from the U.S. Government and the export authority of your entity's jurisdiction. You are responsible for ensuring that you have any required approvals prior to export.

## Regulatory and compliance



**Manufacturer**  
**Hyland Software, Inc.**  
12919 Earhart Ave  
Auburn, CA 95602 USA

**Support**  
Telephone: 1-844-535-1404  
Web: <https://community.hyland.com>

## General warnings and precautions



To ensure patient safety and adhere to HIPAA Privacy Rule involving patient health information, the Healthcare provider shall assign an unique Accession Number and Medical Record Number (MRN) to identify individual patient's health record affiliated with the patient to prevent erroneous medical data and duplicate identification with other patients.

# Symbols glossary



ISO - 15223-1 - Medical devices - Symbols to be used with medical device labels, labeling, and information to be supplied.

3082 - Manufacturer: Indicates the medical device manufacturer.



ISO - 15223-1 - Medical devices - Symbols to be used with medical device labels, labeling, and information to be supplied.

Authorized representative in European Community: Indicates the authorized representative in the European union. Symbol is accompanied by the name and address of the authorized representative adjacent to the symbol.



ISO - 15223-1 - Medical devices - Symbols to be used with medical device labels, labeling, and information to be supplied.

2497 - Date of Manufacture: Indicates the date when the medical device was manufactured.



ISO - 15223-1 - Medical devices - Symbols to be used with medical device labels, labeling, and information to be supplied.

2493 - Catalog number: Indicates the manufacturer's catalog number so the medical device can be identified.



ISO - 15223-1 - Medical devices - Symbols to be used with medical device labels, labeling, and information to be supplied.

2498 - Serial number: Indicates the manufacturer's serial number so that a specific medical device can be identified.



ISO - 15223-1 - Medical devices - Symbols to be used with medical device labels, labeling, and information to be supplied.

1641 - Consult instructions for use: Indicates the need for the user to consult the instructions for use.



ISO - 15223-1 - Medical devices - Symbols to be used with medical device labels, labeling, and information to be supplied.

0434A - Caution: Indicates the need for the user to consult the instructions for use for important information such as warnings and cautions.



ISO - 15223-1 - Medical devices - Symbols to be used with medical device labels, labeling, and information to be supplied.

2610 - Patient number: Indicates a unique number associated with an individual patient.



Quantity: Indicates the number of units.



Directive 2012/19/EU on waste electrical and electronic equipment (WEEE).



ISO/DIS ISO 15223-1 - Medical devices - Symbols to be applied to be used with medical device labels, and information to be supplied.

Indicates the item is a medical device.

# Table of Contents

- Documentation Notice ..... 2
- Regulatory and compliance ..... 3
- General warnings and precautions ..... 3
- Symbols glossary ..... 4
- What is Gear View QC? ..... 7
- Gear View QC 3.2 (March 2025) ..... 7
  - Features ..... 7
  - Updates ..... 7

## What is Gear View QC?

The Gear View QC client is a PACS QC program that you use to correct common mistakes in DICOM image data. It provides functionality to view, import, edit, print, and send DICOM studies to PACS/EHR. QC function includes editing patient demographics and study details, reassigning studies and series, and editing DICOM fields at the image, series, study and patient level.

Gear View QC is intended for use by PACS administrators, physicians, and clinical staff. Users must understand the DICOM image format, as well as the clinical application of the images they modify. Gear View QC is not labeled for diagnostic use. Gear View QC is not intended for the long-term storage or management of patient information or data for patient monitoring. Verify data integrity before saving changes and deleting any original information from your PACS.

## Gear View QC 3.2 (March 2025)

### Features

This release includes the following features.

- The unique identifiers (UIDs) for studies can now be longer than 64 characters.

### Updates

This release includes the following updates.

- Resolved an issue with pixelation in high-contrast images.
- Resolved an issue where the system would not return any results from a patient search.
- Resolved an issue with the image quality of certain CT images.
- Resolved an issue with querying from sources.
- Resolved an issue with the Study ID tags during reconciliation.
- Resolved an issue that limited usernames to 16 characters.
- Resolved an unhandled exception when right-clicking on a series.