



# MDR Video Touch

## Installation and Setup Guide

PG11024 Rev004

## Regulations and Compliance

<b>Australian Sponsor:</b> Emergo Australia Level 20 Tower II Darling Park 201 Sussex Street Sydney, NSW 2000 Australia	<b>Brazil Sponsor:</b> MDR Video Touch Software Registro Anvisa nº: 80117580277 Responsável Técnico: Luiz Levy Cruz Martins CRF- 42415 IMPORTADOR: Emergo Brazil Import Importação e Distribuição de Produtos Médicos Hospitalares Ltda. Avenida Francisco Matarazzo, 1.752, Salas 502/503, Água Branca, São Paulo-SP, CEP - 05001-200 CNPJ: 04.967.408/0001-98 e-mail: brazilvigilance@ul.com		
<b>EU Sponsor:</b> <table border="1"><tr><td><b>EC</b></td><td><b>REP</b></td></tr></table> <b>EMERGO EUROPE</b> Prinsessegracht 20 2514 AP The Hague The Netherlands	<b>EC</b>	<b>REP</b>	
<b>EC</b>	<b>REP</b>		



**Hyland LLC**

12919 Earhart Ave  
Auburn, CA 95602 USA

**Tel: 1-844-535-1404**

**Email: TS\_PACSGEAR@hyland.com**

© 2018 Hyland. Hyland and the Hyland logo are trademarks of Hyland LLC, registered in the United States and/or other countries. All other trademarks are the property of their respective owners.

# Table of Contents

<b>Regulations and Compliance .....</b>	<b>2</b>
<b>What is the MDR Video Touch? .....</b>	<b>4</b>
<b>License the MDR Video Touch .....</b>	<b>4</b>
<b>Log in to the MDR Video Touch .....</b>	<b>4</b>
<b>Change the administrator password .....</b>	<b>4</b>
<b>Update the MDR Video Touch firmware .....</b>	<b>5</b>

## What is the MDR Video Touch?

The MDR Video Touch is a hardware device that captures and sends high resolution images and video from a wide range of modalities to PACS/VNA. It provides image and video capture and transfer at the point of care without the need for a workstation.

The MDR Video Touch has a built-in server that handles security, configuration, and the flow of data between the modalities and destination DICOM systems. However, the MDR Video Touch can also connect to an external instance of Pacsgear Core Server to handle these functions.

## License the MDR Video Touch

**Prerequisite** Obtain a license key by sending an email to [key-pacsgear@kofax.com](mailto:key-pacsgear@kofax.com). In the body of the email, type the following information: Hospital Name, City, State, Country/Region, Product Name, System ID, and Department/Workstation.

You must license the MDR Video Touch before you can begin using it. To license the MDR Video Touch, complete the following steps.

1. From a web browser, go to <http://localhost/pgapps>.

**Note:** If not logging in from the server computer, substitute *localhost* with the IP address of the server computer.

2. On the **License** page, in the **System ID** box, note the system ID.
3. In the **License key** box, type your license key.
4. To install a demo, select the **Enable demo** check box and enter the expiration date in the **Expiration** box.
5. Click **Save**.

## Log in to the MDR Video Touch

To log in to the MDR Video Touch, complete the following steps.

1. Turn on the MDR Video Touch.
2. Optional. To connect to Pacsgear Core Server, complete the following substeps.
  1. On the login screen, click **Settings**.
  2. In the **Server Settings** dialog box, type the IP address of the computer hosting Pacsgear Core Server.
  3. If the server requires a secure connection, select the **Secure (https)** check box.
  4. Click **Save**.
3. On the login screen, log in as `admin`, with password `pacsgear1`.

## Change the administrator password

To increase security, you can change the administrator password by completing the following steps.

1. On the main screen, click **Settings** and click **Server Config**.
2. In the browser, on the **Log in** page, log in as `admin`, with password `pacsgear1`, and then click **Log In**.
3. On the menu bar, click **Settings** and select **Users**.

4. Under **Restrict access permissions to**, click **Local Users**.
5. In the list of users, click the **admin** user name.
6. In the **Add/Edit User** dialog box, type a new password in the **Password** and **Confirm password** boxes.
7. Select the **Administrator** check box and click **Save**.

## Update the MDR Video Touch firmware

Updates to the MDR Video Touch firmware may become available after shipment from the factory. To update the firmware, complete the following steps.

1. Download the zipped upgrade file onto a USB flash drive with at least 512 MB of free space.
2. Turn off the power to the MDR Video Touch, insert the USB flash drive, and then turn the power back on.
3. Tap **Settings**, and in the lower right corner of the screen, tap the hidden button.
4. On the **Utilities** form, tap **Upgrade Software**.
5. When the **MDR Software Upgrade** message appears, tap **Yes**.

**Note:** During the upgrade, several message boxes may appear, but do not tap OK. The message boxes disappear automatically.

6. When the MDR Video Touch restarts and the **Remove upgrade media** message appears, remove the USB flash drive and tap **OK**.