



# MDR Video Touch

## Release Notes

PG11025 Rev004

## Regulations and Compliance

<b>Australian Sponsor:</b> Emergo Australia Level 20 Tower II Darling Park 201 Sussex Street Sydney, NSW 2000 Australia	<b>Brazil Sponsor:</b> MDR Video Touch Software Registro Anvisa nº: 80117580277 Responsável Técnico: Luiz Levy Cruz Martins CRF- 42415 IMPORTADOR: Emergo Brazil Import Importação e Distribuição de Produtos Médicos Hospitalares Ltda. Avenida Francisco Matarazzo, 1.752, Salas 502/503, Água Branca, São Paulo-SP, CEP - 05001-200 CNPJ: 04.967.408/0001-98 e-mail: brazilvigilance@ul.com		
<b>EU Sponsor:</b> <table border="1"><tr><td><b>EC</b></td><td><b>REP</b></td></tr></table> <b>EMERGO EUROPE</b> Prinsessegracht 20 2514 AP The Hague The Netherlands	<b>EC</b>	<b>REP</b>	
<b>EC</b>	<b>REP</b>		



**Hyland LLC**

12919 Earhart Ave  
Auburn, CA 95602 USA

**Tel: 1-844-535-1404**

**Email: TS\_PACSGEAR@hyland.com**

© 2018 Hyland. Hyland and the Hyland logo are trademarks of Hyland LLC, registered in the United States and/or other countries. All other trademarks are the property of their respective owners.

# Table of Contents

**Regulations and Compliance** ..... 2

**What is the MDR Video Touch?** ..... 4

**MDR Video Touch 1.21 (December 2018)** ..... 4

    Features ..... 4

    Updates ..... 4

**MDR Video Touch 1.2 (November 2018)** ..... 4

    Features ..... 4

    Updates ..... 4

**MDR Video Touch 1.1.0 (August 2017)** ..... 4

    Features ..... 4

    Updates ..... 5

## What is the MDR Video Touch?

The MDR Video Touch is a hardware device that captures and sends high resolution images and video from a wide range of modalities to PACS/VNA. It provides image and video capture and transfer at the point of care without the need for a workstation.

The MDR Video Touch has a built-in server that handles security, configuration, and the flow of data between the modalities and destination DICOM systems. However, the MDR Video Touch can also connect to an external instance of Pacsgear Core Server to handle these functions.

## MDR Video Touch 1.21 (December 2018)

### Features

- Worklists are now available offline.
- The system now displays a warning alert when the user presses the Start Exam button without entering the patient's demographics.

### Updates

- Resolved an issue on the Image Edit screen that prevented the user from canceling exams containing images.
- Resolved an issue where the MDR-VT box incorrectly rebooted even when there were no profiles to process.

## MDR Video Touch 1.2 (November 2018)

### Features

- You can now save images in bitmap (BMP) format.
- Added support for international languages.
- When a Send job from the client to the Service fails, the system now resends the metadata for the job on retry.
- Added a Multiple Destinations menu when sending a study to the destination archive.

### Updates

- Resolved an issue that caused the system sound to unmute after a reboot.
- Resolved an issue that caused the system to display a new Capture screen after the user pressed the Enter or space bar keys.
- Resolved an issue that caused an unhandled exception if a lookup source contained an incorrect date/time format.

## MDR Video Touch 1.1.0 (August 2017)

### Features

- MDR-VT now supports capturing video in AVI format.

## Updates

- The caret '^' character is now supported in the MRN field.
- The Numeric VKB now has a '-' key.
- Fixed an issue where UI distortion occurs on the Record form
- The Referring Physician name tag (0008,0090) value now properly passes to the PACS archive
- Fixed an unhandled exception when tapping Alt key on the virtual keyboard