



Media Writer

Installation Guide

LX-DOC-MW5.1.9-IN-EN-REVB

Regulations and Compliance

Australian Sponsor: Emergo Australia Level 20 Tower II Darling Park 201 Sussex Street Sydney, NSW 2000 Australia	Brazil Sponsor: Media Writer Software Registro Anvisa nº: 80117589010 Responsável Técnico: Luiz Levy Cruz Martins CRF- 42415 IMPORTADOR: Emergo Brazil Import Importação e Distribuição de Produtos Médicos Hospitalares Ltda. Avenida Francisco Matarazzo, 1.752, Salas 502/503, Água Branca, São Paulo-SP, CEP - 05001-200 - Cidade: Brasília / UF: DF CNPJ: 04.967.408/0001-98 e-mail: brazilvigilance@emergogroup.com		
EU Sponsor: <table border="1"><tr><td>EC</td><td>REP</td></tr></table> EMERGO EUROPE Prinsessegracht 20 2514 AP The Hague The Netherlands	EC	REP	
EC	REP		



Hyland LLC

4309 Hacienda Drive, Suite 500
Pleasanton, CA 94588 USA

Tel: 1-844-535-1404

Email: TS_PACSGEAR@hyland.com

© 2018 Hyland. Hyland and the Hyland logo are trademarks of Hyland LLC, registered in the United States and/or other countries. All other trademarks are the property of their respective owners.

Table of Contents

Regulations and Compliance	2
Download the Media Writer setup program	4
Install the Media Writer service	4
Install the Media Writer client	5
License Media Writer	5
Log in for the first time	5
Install the Media Writer web client	6
Launch the Media Writer web client	6

Download the Media Writer setup program

To download the Media Writer setup program to the host computer, perform one of the following actions.

- From a browser, go to the URL provided by your Lexmark Enterprise Software representative. On the Download page, click the link. If prompted, specify a destination folder and then click OK.
- Insert the Media Writer installation disc into the CD-ROM drive of the host computer. In Windows, copy the installation folder from the CD-ROM to the host computer.

Install the Media Writer service

To install the Media Writer service, complete the following steps.

1. Navigate to the *PACSGEAR Media Server for MediaWriter* folder, and then double-click *setup.exe*.
2. If the Windows UAC prompt appears, click **Yes**.
3. In the language selection dialog box, click a language in the list and then click **OK**.
4. In the **Welcome to the InstallShield Wizard for MediaWriter Server** dialog box, click **Next**.
5. In the **License Agreement** page, scroll to the bottom of the license agreement, and then click **Yes**.
6. In the **Choose Destination Location** dialog box, browse for a new location if necessary, and then click **Next**.
7. In the **Select Features** page, perform one of the following actions.
 - To install a new SQL database, select all features, and then click **Next**.
 - To use an existing local or remote database, clear the **Microsoft SQL Server 2008 Express** check box, click **Next**.
8. Optional. If you chose an existing database above, complete the following substeps.
 1. In the **Host** box, type the host or instance of the database server you want to use.
 2. In the **User name** and **Password** boxes, type server credentials.
 3. In the **Database** box, type the name of the database you want to use.
 4. Click **Continue**.
9. If no message appears, skip to the next step. If the message, The installation process has detected that a compatible version of SQL Server has already been installed appears, click **OK**, and then complete the following substeps.
 1. In the **Database server that you are installing** to list, select or browse for a database.
 2. Under **Connect using**, click **Windows authentication**.
 3. Optional. To use a local SQL server, click **SQL Server authentication**, and then type your SQL login ID and password.
 4. In the **Name of database catalog** box, browse for the catalog.
10. Click **Next**.
11. In the **Ready to Install the Program** page, click **Install**.
InstallShield begins installing Gear View QC Server. If prompted, follow the on-screen instructions to install any missing requirements or prerequisites.
12. In the **InstallShield Wizard Complete** page, click **Yes**, I want to restart my computer now, and then click **Finish**.

Install the Media Writer client

To install the Media Writer client, complete the following steps.

1. Navigate to the *MediaWriter Client* folder, and then double-click *setup.exe*.
2. If the Windows UAC prompt appears, click **Yes**.
3. In the **Select the language** dialog box, click a language in the list and then click **OK**.
4. In the **Welcome to the InstallShield Wizard for MediaWriter Client** dialog box, click **Next**.
5. In the **License Agreement** page, scroll to the bottom of the license agreement, and then click **Yes**.
6. In the **Choose Destination Location** dialog box, browse for a new location if necessary, and then click **Next**.
7. In the **Select Features** page, click **Next**.
8. In the **Ready to Install the Program** page, click **Install**.
InstallShield begins installing Media Writer Client. If prompted, follow the on-screen instructions to install any missing requirements or prerequisites.
9. In the **InstallShield Wizard Complete** page, click **Finish**.

License Media Writer

To license Media Writer, complete the following steps.

1. To launch the Media Writer client program, double-click the MediaWriter desktop icon.
2. In the **MediaWriter License** dialog box, in the **System ID** box, note the system ID.
3. To obtain a license key, send an email to key-pacsgear@kofax.com. In the body of the email, type the following information:
 - Site name
 - City, State/Province
 - Country/Region
 - Product name
 - System ID
 - Department/Workstation
4. When you receive your license key, enter it in the **License key** box.
5. To install a demo, select the **Activate demo** check box. In the **Month**, **Date**, and **Year** boxes, type the expiration date that was included with your license key.
6. Click **OK** to display the **Log On** dialog box.

Log in for the first time

After you complete the licensing process, the Log On dialog box appears. To log in, complete the following steps.

1. In the **User name** box, type `admin`.
2. Leave the **Password** box blank.
3. In the **Server** box, accept the default value of localhost or type the IP address of the PC on which you installed the Media Writer server.
4. In the **Port** box, accept the default value of 8000, type the port number of the PC on which you installed the Media Writer server, and then click **OK**.

Install the Media Writer web client

To install the Media Writer web client, complete the following steps.

1. Navigate to the *Media Writer Web+ClickOnce* folder, and then double-click *setup.exe*.
2. Follow all prompts and then click **Finish** when installation is complete.
3. Restart the computer.

Launch the Media Writer web client

To launch the Media Writer Web Client, complete the following steps.

1. Launch a web browser.
2. In the address box, type `http://localhost/mediawriter`, and then press *ENTER*.