



# Media Writer

## Alternate Viewer Installation Guide

DOC-MW-AV-IN-EN-REVA

## Regulations and Compliance

<b>Australian Sponsor:</b> Emergo Australia Level 20 Tower II Darling Park 201 Sussex Street Sydney, NSW 2000 Australia	<b>Brazil Sponsor:</b> Media Writer Software Registro Anvisa nº: 80117580229 Responsável Técnico: Luiz Levy Cruz Martins CRF- 42415 IMPORTADOR: Emergo Brazil Import Importação e Distribuição de Produtos Médicos Hospitalares Ltda. Avenida Francisco Matarazzo, 1.752, Salas 502/503, Água Branca, São Paulo-SP, CEP - 05001-200 CNPJ: 04.967.408/0001-98 e-mail: brazilvigilance@ul.com		
<b>EU Sponsor:</b> <table border="1"><tr><td>EC</td><td>REP</td></tr></table> <b>EMERGO EUROPE</b> Prinsessegracht 20 2514 AP The Hague The Netherlands	EC	REP	
EC	REP		



**Hyland LLC**

4309 Hacienda Drive, Suite 500  
Pleasanton, CA 94588 USA

**For support, contact:**

**Tel: 1-844-535-1404**

**Email: [Key-pacsgear@hyland.com](mailto:Key-pacsgear@hyland.com)**

**Web: <https://community.hyland.com>**

© 2019 Hyland. Hyland and the Hyland logo are trademarks of Hyland LLC, registered in the United States and/or other countries. All other trademarks are the property of their respective owners.

## Symbols used in this document



European Economic Area certification mark



ISO 7000:3500 - Electronic Instructions For Use

To indicate on product or product packaging that relevant information for use of the product is available in electronic form rather than, or in addition to, printed paper form.



ISO 7000:3082 - Manufacturer

To identify the manufacturer of a product.



ISO 7000: 0434B – Caution

To indicate that caution is necessary when operating the device or control close to where the symbol is placed, or to indicate that the current situation needs operator awareness or operator action in order to avoid undesirable consequences.

# Table of Contents

**Regulations and Compliance** ..... 2

    Symbols used in this document ..... 3

**About viewers for Media Writer** ..... 5

**Install and alternate viewer** ..... 5

**Edit the viewer configuration file** ..... 5

## About viewers for Media Writer

Media Writer comes with a viewer called Gear View Basic that you can include on burned media so that patients can view DICOM studies. If you prefer to include a different viewer, you must first install it on the Media Writer server computer and then add it as part of your Media Writer installation. The alternate viewers that are available are eFilm 3.4, MIMVista, and ThreePalm.

## Install and alternate viewer

To install an alternate viewer, complete the following steps.

### Prerequisite

Obtain the USB flash drive containing the viewer files and configuration files from your Lexmark sales representative.

1. On the server computer for Media Writer, create a new folder called *Original* under the *C:/Program Files (x86)/Pacsgear/Media Writer/Service/Config* folder.
2. Move the default *viewers.config* file from the *Config* folder into the *Original* folder.
3. Copy the *xyz\_viewers.config* file from the USB flash drive to *Media Writer/Service/Config*.

**Note:** xyz represents efilm, mim, or tsp depending on which viewer you want to install.

4. Rename the *xyz\_viewers.config* file to *viewers.config*.
5. Copy the appropriate Viewer folder from the USB flash drive to *../Media Writer/Service/Viewers*.
6. If installing the ThreePalm Mammoviewer, copy *postinstall.exe* from the USB flash drive to *../Media Writer/Service*.
7. Close all windows, eject the USB flash drive, and then restart the computer.

## Edit the viewer configuration file

After installing the alternate viewer and configuration file, you must edit the configuration file of the new viewer to make it available from within Media Writer. To edit the viewer configuration file, complete the following steps.

1. In **Control Panel**, click **Administrative Tools**, and then double-click **Services**.
2. Select the Media Writer service in the list, and then click **Stop**.
3. Open the *../Media Writer/Service/Config/viewers.config* file in Notepad.
4. At the bottom of the file, add the text below that corresponds to the viewer you installed.

Viewer	Add to configuration file
eFilm	<pre>&lt;VIEWER&gt; &lt;NAME-&gt;eFilm&lt;/NAME&gt; &lt;PATH&gt;viewer/eFilm&lt;/PATH&gt; &lt;VIEWER_EXE_FILE&gt;efilmLt.exe&lt;/VIEWER_EXE_FILE&gt; &lt;/VIEWER&gt;</pre>

Viewer	Add to configuration file
3Palm Mammo	<pre> &lt;VIEWER&gt; &lt;NAME-&gt;MAMMO TPS&lt;/NAME&gt; &lt;PATH&gt;<i>[relative path to the viewer folder]</i>&lt;/PATH&gt; &lt;VIEWER_EXE_FILE&gt;<i>[full name of the viewer executable file]</i>&lt;/VIEWER_EXE_ FILE&gt; &lt;/VIEWER&gt; &lt;VIEWER_LICGEN_EXE&gt;postinstall.exe&lt;/VIEWER_LICGEN_EXE&gt; </pre>
MIMVista	<pre> &lt;VIEWER&gt; &lt;NAME-&gt;MIMVISTA&lt;/NAME&gt; &lt;PATH&gt;<i>[relative path to the viewer folder]</i>&lt;/PATH&gt; &lt;VIEWER_EXE_FILE&gt;<i>[full name of the viewer executable file]</i>&lt;/VIEWER_EXE_ FILE&gt; &lt;/VIEWER&gt; &lt;VIEWER_LICGEN_EXE&gt;mvprep.exe&lt;/VIEWER_LICGEN_EXE&gt; </pre>

5. Click **Save** and close Notepad.
6. Restart the Media Writer service