



MediaWriter

D200/D100 Installation and Setup Guide

DOC-MW-EPSON-IN-EN-REVA

Regulations and Compliance

Australian Sponsor: Emergo Australia Level 20 Tower II Darling Park 201 Sussex Street Sydney, NSW 2000 Australia	Brazil Sponsor: Media Writer Software Registro Anvisa nº: 80117580229 Responsável Técnico: Luiz Levy Cruz Martins CRF- 42415 IMPORTADOR: Emergo Brazil Import Importação e Distribuição de Produtos Médicos Hospitalares Ltda. Avenida Francisco Matarazzo, 1.752, Salas 502/503, Água Branca, São Paulo-SP, CEP - 05001-200 CNPJ: 04.967.408/0001-98 e-mail: brazilvigilance@ul.com		
EU Sponsor: <table border="1"><tr><td>EC</td><td>REP</td></tr></table> EMERGO EUROPE Prinsessegracht 20 2514 AP The Hague The Netherlands	EC	REP	
EC	REP		



Hyland LLC

4309 Hacienda Drive, Suite 500
Pleasanton, CA 94588 USA

For support, contact:

Tel: 1-844-535-1404

Email: Key-pacsgear@hyland.com

Web: <https://community.hyland.com>

© 2019 Hyland. Hyland and the Hyland logo are trademarks of Hyland LLC, registered in the United States and/or other countries. All other trademarks are the property of their respective owners.

Symbols used in this document



European Economic Area certification mark



ISO 7000:3500 - Electronic Instructions For Use

To indicate on product or product packaging that relevant information for use of the product is available in electronic form rather than, or in addition to, printed paper form.



ISO 7000:3082 - Manufacturer

To identify the manufacturer of a product.



ISO 7000: 0434B – Caution

To indicate that caution is necessary when operating the device or control close to where the symbol is placed, or to indicate that the current situation needs operator awareness or operator action in order to avoid undesirable consequences.

Table of Contents

Regulations and Compliance	2
Symbols used in this document	3
Install and license Media Writer	5
Install EPSON Total Disc Maker	5
Configure the EPSON	5
About publish modes	6
Install EPSON TD Bridge	6
Configure EPSON TD Bridge	6
Settings	7
<i>EPSON settings</i>	7
<i>EPSON TD Bridge settings</i>	7

Install and license Media Writer

To install the Media Writer Server, and to install and license Media Writer Client, see the Media Writer Installation and Setup Guide.

Install EPSON Total Disc Maker

Prerequisite Remove any previously installed CD/DVD burning applications, such as Nero and Easy CD Creator, and disable any firewalls installed on the system such as Windows Firewall, Norton Personal Firewall, or Norton Internet Security. Set up the EPSON PP-50 or PP-100/PP-100II Discproducer according to the instructions from Hyland that came with the product.

To install EPSON Total Disc Maker, complete the following steps.

1. Disconnect the USB cable of the EPSON from the computer and turn on the EPSON.
2. On the **EPSON TOTAL DISC MAKER 612/EPSON TD BRIDGE 612** DVD, navigate to the **TDMaker 612** folder and double-click **setup.exe**.

Note: If the **User Account Control** dialog box appears, click **Yes**.

3. On the **InstallShield Welcome** page, click **Next**.
4. On the **License Agreement** dialog box, select the **I accept the terms in the license agreement**, click **Next**, and then click **Next** again.
5. On the **Model selection** page, select the check box for your burner model and click **Next**.

Note:

- For the EPSON PP-100 or PP-100II, select EPSON PP-100II / PP-100 (USB Connection).
- For the EPSON PP-50, select EPSON PP-50BD / PP-50 / PP-7050BD (USB Connection).

6. On the **Setup Type** page, select **Complete** and click **Next**.
7. On the **Ready to Install Program** page, click **Install**.
8. When the **Utilities Setup** page appears, plug the EPSON's USB cable into the PC.
Windows installs the device drivers automatically.
9. On the **InstallShield Wizard Complete** page, click **Finish**.

Configure the EPSON

To configure the EPSON Total Disc Maker, complete the following steps.

1. On the Windows **Start** menu, navigate to **EPSON Total Disc Setup**.
The Total Disc Setup screen and the Common Setting for Publishing dialog box appear.
2. In the **Common Setting for Publishing** dialog box, click **OK**.
3. On the **Total Disc Setup** screen, double click **PP-100 1** or **PP-50 1**.
4. In the **Properties** dialog box, on the **General** tab, enter the settings described under "EPSON Settings" (see "Settings").
5. Click **Apply** and click **OK**.
6. On the **Total Disc Setup** screen, confirm that the `pacsgear` device appears.

About publish modes

The publish mode you select for your configuration can depend on factors such as your printer or media type. The different publish modes available are Batch, External Output, Standard, and Low Volume.

Batch Mode (Recommended)

Use when there is only one type of medium in both trays 1 and 2, and when you want to burn 50 or more discs. In Batch mode, the printer places completed discs in tray 3.

External Output Mode

Use this mode when using two types of media such as a CD-R in tray 1 and a DVD-R 4.7GB in tray 2. The maximum capacity of tray 4 is eight discs. In this mode, the printer places completed discs in tray 4. When tray 4 is full, Media Writer places burn jobs on hold until you empty the tray.

Standard Mode

Use this mode when there is only one type of medium in tray 1. In this mode, the printer places completed discs in tray 4 until the tray is full, and then places them in tray 2.

Low Volume Mode (PP-50 only)

Use this mode to use trays 1 and 2 for input, and tray 3 for completed discs. You can load different types of discs (CD or DVD) into trays 1 and 2. This mode is good for creating small batches of discs at once.

Install EPSON TD Bridge

To install EPSON TD Bridge, complete the following steps.

1. On the **EPSON TOTAL DISC MAKER 612 / EPSON TD BRIDGE 612** DVD, navigate to the **Total Disc Bridge 612\setup** folder and double-click **setup.exe**.

Note: If the **User Account Control** dialog box appears, click **Yes**.

2. On the **InstallShield Welcome** page, click **Next**.
3. On the **License Agreement** page, click **I accept the terms of the license agreement** and click **Next**.
4. On the **Choose Destination Location** page, click **Next**.
5. On the **Select a Monitoring Folder** page, click **Next**.
6. On the **Ready to Install the Program** page, click **Install**.
7. On the **InstallShield Wizard Complete** page, select the **Starts EPSON TD Bridge** check box, and click **Finish**.

Configure EPSON TD Bridge

To configure EPSON TD Bridge, complete the following steps.

1. On the Windows task bar, right-click the **TD Bridge** icon and click **Environment settings**.

Note: If the **User Account Control** dialog box appears, click **Yes**.

2. On the **General** tab, configure the settings under "EPSON TD Bridge Settings" (see "Settings").
3. Click **OK**.

4. In the **Reboot the computer to enable the settings** message box, click **OK**.
5. In Media Writer, set the device type to **EPSON_BURNER**.

Settings

EPSON settings

To configure the EPSON, see "Configure the EPSON" and enter the settings described in the following table.

Setting	PP-100 1	PP-50 1
Name	pacsgear	pacsgear
Publish Mode	External Output mode	Low-volume mode
Stacker 1	CD-R	CD-R
Stacker 2	DVD(+R/-R)	CD-R
Stacker 3	Output Stacker	CD-R
Drive to use	Use Both	
Number of write retries	3	3
Printer Name	EPSON PP-100PRN	EPSON PP-50PRN
Show error mark on discs	Select	Select

EPSON TD Bridge settings

To configure EPSON TD Bridge, see "Configure EPSON TD Bridge" and enter the settings described in the following table.

Item	Setting
Monitoring Folder	C:\EPSON\TDBridge\Orders
TD Bridge Startup Type	Automatic (when starting PC/service)
Default output device	pacsgear
Default output stacker in standard mode	Stacker 3 or Stacker 4
Label Type	CD/DVD Label