



Media Writer
Alternate Viewer Installation Guide

DOC-MW-AV-IN-EN-REVA

Copyright

Information in this document is subject to change without notice. The software described in this document is furnished only under a separate license agreement and may be used or copied only according to the terms of such agreement. It is against the law to copy the software except as specifically allowed in the license agreement. This document or accompanying materials contains certain information which is confidential information of Hyland Software, Inc. and its affiliates, and which is subject to the confidentiality provisions agreed to by you.

All data, names, and formats used in this document's examples are fictitious unless noted otherwise. Complying with all applicable copyright laws is the responsibility of the user. Without limiting the rights under copyright law, no part of this document may be reproduced, stored in or introduced into a retrieval system, or transmitted in any form or by any means (electronic, mechanical, photocopying, recording, or otherwise), or for any purpose, without the express written permission of Hyland Software, Inc. or one of its affiliates.

Hyland®, Hyland Software®, Hyland Healthcare, and Hyland product names are registered and/or unregistered trademarks of Hyland Software, Inc. and its affiliates in the United States and other countries. All other trademarks, service marks, trade names and products of other companies are the property of their respective owners.

© 2022 Hyland Software, Inc. and its affiliates. All rights reserved.

Regulatory and compliance

Australian Sponsor: Emergo Australia Level 20 Tower II Darling Park 201 Sussex Street Sydney, NSW 2000 Australia		Media Writer Software Anvisa n°: 80117580229 Responsável Técnico: Luiz Levy Cruz Martins CRF-SP: 42415 IMPORTADOR: Emergo Brazil Import Importação e Distribuição de Produtos Médicos Hospitalares Ltda. Avenida Francisco Matarazzo, 1.752, Salas 502/503, Água Branca, São Paulo-SP, CEP - 05001-200 CNPJ: 04.967.408/0001-98 e-mail: brazilvigilance@ul.com	
EU Representative:  EMERGO EUROPE Prinsessegracht 20 2514 AP The Hague The Netherlands			



Manufacturer
Hyland LLC
4309 Hacienda Drive,
Suite 500
Pleasanton, CA 94588
USA

Support
Telephone: 1-844-535-1404
Web: <https://community.hyland.com>

General warnings and precautions



To ensure patient safety and adhere to HIPAA Privacy Rule involving patient health information, the Healthcare provider shall assign an unique Accession Number and Medical Record Number (MRN) to identify individual patient's health record affiliated with the patient to prevent erroneous medical data and duplicate identification with other patients.

Symbols glossary



ISO - 15223-1 - Medical devices - Symbols to be used with medical device labels, labeling, and information to be supplied.

3082 - Manufacturer: Indicates the medical device manufacturer.



ISO - 15223-1 - Medical devices - Symbols to be used with medical device labels, labeling, and information to be supplied.

Authorized representative in European Community: Indicates the authorized representative in the European union. Symbol is accompanied by the name and address of the authorized representative adjacent to the symbol.



ISO - 15223-1 - Medical devices - Symbols to be used with medical device labels, labeling, and information to be supplied.

2497 - Date of Manufacture: Indicates the date when the medical device was manufactured.



ISO - 15223-1 - Medical devices - Symbols to be used with medical device labels, labeling, and information to be supplied.

2493 - Catalog number: Indicates the manufacturer's catalog number so the medical device can be identified.



ISO - 15223-1 - Medical devices - Symbols to be used with medical device labels, labeling, and information to be supplied.

2498 - Serial number: Indicates the manufacturer's serial number so that a specific medical device can be identified.



ISO - 15223-1 - Medical devices - Symbols to be used with medical device labels, labeling, and information to be supplied.

1641 - Consult instructions for use: Indicates the need for the user to consult the instructions for use.



ISO - 15223-1 - Medical devices - Symbols to be used with medical device labels, labeling, and information to be supplied.

0434A - Caution: Indicates the need for the user to consult the instructions for use for important information such as warnings and cautions.



ISO - 15223-1 - Medical devices - Symbols to be used with medical device labels, labeling, and information to be supplied.

2610 - Patient number: Indicates a unique number associated with an individual patient.



Medical Devices Directive 2017/745/EC

CE Marking: Indicates that a product complies with applicable European Union regulations.



Quantity: Indicates the number of units.



Directive 2012/19/EU on waste electrical and electronic equipment (WEEE).



ISO/DIS ISO 15223-1 - Medical devices - Symbols to be applied to be used with medical device labels, and information to be supplied.

Indicates the item is a medical device.

Table of Contents

Copyright 2

Regulatory and compliance 3

General warnings and precautions 3

Symbols glossary 4

About viewers for Media Writer 7

Install and alternate viewer 7

Edit the viewer configuration file 7

About viewers for Media Writer

Media Writer comes with a viewer called Gear View Basic that you can include on burned media so that patients can view DICOM studies. If you prefer to include a different viewer, you must first install it on the Media Writer server computer and then add it as part of your Media Writer installation. The alternate viewers that are available are eFilm 3.4, MIMVista, and ThreePalm.

Install and alternate viewer

To install an alternate viewer, complete the following steps.

Prerequisite

Obtain the USB flash drive containing the viewer files and configuration files from your Lexmark sales representative.

1. On the server computer for Media Writer, create a new folder called *Original* under the *C:/Program Files (x86)/Pacsgear/Media Writer/Service/Config* folder.
2. Move the default *viewers.config* file from the *Config* folder into the *Original* folder.
3. Copy the *xyz_viewers.config* file from the USB flash drive to *Media Writer/Service/Config*.

Note: xyz represents efilm, mim, or tsp depending on which viewer you want to install.

4. Rename the *xyz_viewers.config* file to *viewers.config*.
5. Copy the appropriate Viewer folder from the USB flash drive to *../Media Writer/Service/Viewers*.
6. If installing the ThreePalm Mammoviewer, copy *postinstall.exe* from the USB flash drive to *../Media Writer/Service*.
7. Close all windows, eject the USB flash drive, and then restart the computer.

Edit the viewer configuration file

After installing the alternate viewer and configuration file, you must edit the configuration file of the new viewer to make it available from within Media Writer. To edit the viewer configuration file, complete the following steps.

1. In **Control Panel**, click **Administrative Tools**, and then double-click **Services**.
2. Select the Media Writer service in the list, and then click **Stop**.
3. Open the *../Media Writer/Service/Config/viewers.config* file in Notepad.
4. At the bottom of the file, add the text below that corresponds to the viewer you installed.

Viewer	Add to configuration file
eFilm	<pre><VIEWER> <NAME->eFilm</NAME> <PATH>viewer/eFilm</PATH> <VIEWER_EXE_FILE>efilmLt.exe</VIEWER_EXE_FILE> </VIEWER></pre>

Viewer	Add to configuration file
3Palm Mammo	<pre> <VIEWER> <NAME->MAMMO TPS</NAME> <PATH><i>[relative path to the viewer folder]</i></PATH> <VIEWER_EXE_FILE><i>[full name of the viewer executable file]</i></VIEWER_EXE_ FILE> </VIEWER> <VIEWER_LICGEN_EXE>postinstall.exe</VIEWER_LICGEN_EXE > </pre>
MIMVista	<pre> <VIEWER> <NAME->MIMVISTA</NAME> <PATH><i>[relative path to the viewer folder]</i></PATH> <VIEWER_EXE_FILE><i>[full name of the viewer executable file]</i></VIEWER_EXE_ FILE> </VIEWER> <VIEWER_LICGEN_EXE>mvprep.exe</VIEWER_LICGEN_EXE > </pre>

5. Click **Save** and close Notepad.
6. Restart the Media Writer service