



Media Writer D200/D100 Installation Guide

DOC-MW-EPSON-IN-EN-REVA

Copyright

Information in this document is subject to change without notice. The software described in this document is furnished only under a separate license agreement and may be used or copied only according to the terms of such agreement. It is against the law to copy the software except as specifically allowed in the license agreement. This document or accompanying materials contains certain information which is confidential information of Hyland Software, Inc. and its affiliates, and which is subject to the confidentiality provisions agreed to by you.

All data, names, and formats used in this document's examples are fictitious unless noted otherwise. Complying with all applicable copyright laws is the responsibility of the user. Without limiting the rights under copyright law, no part of this document may be reproduced, stored in or introduced into a retrieval system, or transmitted in any form or by any means (electronic, mechanical, photocopying, recording, or otherwise), or for any purpose, without the express written permission of Hyland Software, Inc. or one of its affiliates.

Hyland®, Hyland Software®, Hyland Healthcare, and Hyland product names are registered and/or unregistered trademarks of Hyland Software, Inc. and its affiliates in the United States and other countries. All other trademarks, service marks, trade names and products of other companies are the property of their respective owners.

© 2022 Hyland Software, Inc. and its affiliates. All rights reserved.

Regulatory and compliance

Australian Sponsor: Emergo Australia Level 20 Tower II Darling Park 201 Sussex Street Sydney, NSW 2000 Australia		Media Writer Software Anvisa n°: 80117580229 Responsável Técnico: Luiz Levy Cruz Martins CRF-SP: 42415 IMPORTADOR: Emergo Brazil Import Importação e Distribuição de Produtos Médicos Hospitalares Ltda. Avenida Francisco Matarazzo, 1.752, Salas 502/503, Água Branca, São Paulo-SP, CEP - 05001-200 CNPJ: 04.967.408/0001-98 e-mail: brazilvigilance@ul.com	
EU Representative:  EMERGO EUROPE Prinsessegracht 20 2514 AP The Hague The Netherlands			



Manufacturer
Hyland LLC
4309 Hacienda Drive,
Suite 500
Pleasanton, CA 94588
USA

Support
Telephone: 1-844-535-1404
Web: <https://community.hyland.com>

General warnings and precautions



To ensure patient safety and adhere to HIPAA Privacy Rule involving patient health information, the Healthcare provider shall assign an unique Accession Number and Medical Record Number (MRN) to identify individual patient's health record affiliated with the patient to prevent erroneous medical data and duplicate identification with other patients.

Symbols glossary



ISO - 15223-1 - Medical devices - Symbols to be used with medical device labels, labeling, and information to be supplied.

3082 - Manufacturer: Indicates the medical device manufacturer.



ISO - 15223-1 - Medical devices - Symbols to be used with medical device labels, labeling, and information to be supplied.

Authorized representative in European Community: Indicates the authorized representative in the European union. Symbol is accompanied by the name and address of the authorized representative adjacent to the symbol.



ISO - 15223-1 - Medical devices - Symbols to be used with medical device labels, labeling, and information to be supplied.

2497 - Date of Manufacture: Indicates the date when the medical device was manufactured.



ISO - 15223-1 - Medical devices - Symbols to be used with medical device labels, labeling, and information to be supplied.

2493 - Catalog number: Indicates the manufacturer's catalog number so the medical device can be identified.



ISO - 15223-1 - Medical devices - Symbols to be used with medical device labels, labeling, and information to be supplied.

2498 - Serial number: Indicates the manufacturer's serial number so that a specific medical device can be identified.



ISO - 15223-1 - Medical devices - Symbols to be used with medical device labels, labeling, and information to be supplied.

1641 - Consult instructions for use: Indicates the need for the user to consult the instructions for use.



ISO - 15223-1 - Medical devices - Symbols to be used with medical device labels, labeling, and information to be supplied.

0434A - Caution: Indicates the need for the user to consult the instructions for use for important information such as warnings and cautions.



ISO - 15223-1 - Medical devices - Symbols to be used with medical device labels, labeling, and information to be supplied.

2610 - Patient number: Indicates a unique number associated with an individual patient.



Medical Devices Directive 2017/745/EC

CE Marking: Indicates that a product complies with applicable European Union regulations.



Quantity: Indicates the number of units.



Directive 2012/19/EU on waste electrical and electronic equipment (WEEE).



ISO/DIS ISO 15223-1 - Medical devices - Symbols to be applied to be used with medical device labels, and information to be supplied.

Indicates the item is a medical device.

Table of Contents

Copyright	2
Regulatory and compliance	3
General warnings and precautions	3
Symbols glossary	4
Install and license Media Writer	7
Install EPSON Total Disc Maker	7
Upgrade EPSON Total Disc Maker software	7
Configure the EPSON Total Disc Maker	8
About publish modes	8
Install EPSON TD Bridge	8
Upgrade EPSON TD Bridge software	9
Configure EPSON TD Bridge	9
Settings	9
<i>EPSON settings</i>	9
<i>EPSON TD Bridge settings</i>	10

Install and license Media Writer

To install the Media Writer Server, and to install and license Media Writer Client, see the Media Writer Installation and Setup Guide.

Install EPSON Total Disc Maker

Prerequisite Remove any previously installed CD/DVD burning applications, such as Nero and Easy CD Creator, and disable any firewalls installed on the system such as Windows Firewall, Norton Personal Firewall, or Norton Internet Security. Set up the EPSON PP-50 or PP-100/PP-100II Discproducer according to the instructions from Hyland that came with the product.

To install EPSON Total Disc Maker, complete the following steps.

1. Disconnect the USB cable of the EPSON from the computer and turn on the EPSON.
2. On the **EPSON TOTAL DISC MAKER 612/EPSON TD BRIDGE 612** DVD, navigate to the **TDMaker 612** folder and double-click **setup.exe**.

Note: If the **User Account Control** dialog box appears, click **Yes**.

3. On the **InstallShield Welcome** page, click **Next**.
4. On the **License Agreement** dialog box, select the **I accept the terms in the license agreement**, click **Next**, and then click **Next** again.
5. On the **Model selection** page, select the check box for your burner model and click **Next**.

Note:

- For the EPSON PP-100, PP-100II or PP-100III select EPSON PP-100II / PP-100/PP-100III (USB Connection).
- For the EPSON PP-50, select EPSON PP-50BD / PP-50 / PP-7050BD (USB Connection).

6. On the **Setup Type** page, select **Complete** and click **Next**.
7. On the **Ready to Install Program** page, click **Install**.
8. When the **Utilities Setup** page appears, plug the EPSON's USB cable into the PC.
Windows installs the device drivers automatically.
9. On the **InstallShield Wizard Complete** page, click **Finish**.

Upgrade EPSON Total Disc Maker software



Before installing the latest EPSON Total Disc Maker software, we highly recommend that you manually uninstall the current version.

To upgrade EPSON Total Disc Maker software, complete the following steps.

1. On your device, login as an administrator and open the **Add or Remove Programs** control panel.
 1. Select **EPSON TD Total Disc Maker** and then click **Uninstall**.
 2. If you want to delete log files, click **YES** when prompted.
2. After the installation is complete, delete the following folders.

- C:\Program Files (x86)\Epson
 - The working directory you defined during installation. For example, C:\EPSON
3. Reboot your PC.
 4. Install EPSON TD Maker 7.00 or later.
 5. Reboot your device.

Configure the EPSON Total Disc Maker

To configure the EPSON Total Disc Maker, complete the following steps.

1. On the Windows **Start** menu, navigate to **EPSON Total Disc Setup**.
The Total Disc Setup screen and the Common Setting for Publishing dialog box appear.
2. In the **Common Setting for Publishing** dialog box, click **OK**.
3. On the **Total Disc Setup** screen, double click **PP-100 1** or **PP-50 1**.
4. In the **Properties** dialog box, on the **General** tab, enter the settings described under "EPSON Settings" (see "Settings").
5. Click **Apply** and click **OK**.
6. On the **Total Disc Setup** screen, confirm that the `pacsgear` device appears.

About publish modes

The publish mode you select for your configuration can depend on factors such as your printer or media type. The different publish modes available are Batch, External Output, Standard, and Low Volume.

Batch Mode (Recommended)

Use when there is only one type of medium in both trays 1 and 2, and when you want to burn 50 or more discs. In Batch mode, the printer places completed discs in tray 3.

External Output Mode

Use this mode when using two types of media such as a CD-R in tray 1 and a DVD-R 4.7GB in tray 2. The maximum capacity of tray 4 is eight discs. In this mode, the printer places completed discs in tray 4. When tray 4 is full, Media Writer places burn jobs on hold until you empty the tray.

Standard Mode

Use this mode when there is only one type of medium in tray 1. In this mode, the printer places completed discs in tray 4 until the tray is full, and then places them in tray 2.

Low Volume Mode (PP-50 only)

Use this mode to use trays 1 and 2 for input, and tray 3 for completed discs. You can load different types of discs (CD or DVD) into trays 1 and 2. This mode is good for creating small batches of discs at once.

Install EPSON TD Bridge

To install EPSON TD Bridge, complete the following steps.

1. On the **EPSON TOTAL DISC MAKER 612 / EPSON TD BRIDGE 612 DVD**, navigate to the **Total Disc Bridge 612\setup** folder and double-click **setup.exe**.

Note: If the **User Account Control** dialog box appears, click **Yes**.

2. On the **InstallShield Welcome** page, click **Next**.
3. On the **License Agreement** page, click **I accept the terms of the license agreement** and click **Next**.
4. On the **Choose Destination Location** page, click **Next**.
5. On the **Select a Monitoring Folder** page, click **Next**.
6. On the **Ready to Install the Program** page, click **Install**.
7. On the **InstallShield Wizard Complete** page, select the **Starts EPSON TD Bridge** check box, and click **Finish**.

Upgrade EPSON TD Bridge software



Before installing the latest TD Bridge software, we highly recommend that you manually uninstall the current version.

To upgrade EPSON TD Bridge software, complete the following steps.

1. On your device, login as an administrator and open the **Add or Remove Programs** control panel.
 1. Select **EPSON TD Bridge** and then click **Uninstall**.
 2. If you want to delete log files, click **YES** when prompted.
2. After the uninstall is complete, delete the following folders.
 - C:\Program Files (x86)\Epson
 - The working directory you defined during installation. For example, C:\EPSON
3. Reboot your PC.
4. Install EPSON TD Bridge 7.00 or later.
5. Reboot your device.

Configure EPSON TD Bridge

To configure EPSON TD Bridge, complete the following steps.

1. On the Windows task bar, right-click the **TD Bridge** icon and click **Environment settings**.

Note: If the **User Account Control** dialog box appears, click **Yes**.

2. On the **General** tab, configure the settings under "EPSON TD Bridge Settings" (see "Settings").
3. Click **OK**.
4. In the **Reboot the computer to enable the settings** message box, click **OK**.
5. In Media Writer, set the device type to **EPSON_BURNER**.

Settings

EPSON settings

To configure the EPSON, see "Configure the EPSON" and enter the settings described in the following table.

Setting	PP-100 1	PP-50 1
Name	pacsgear	pacsgear
Publish Mode	External Output mode	Low-volume mode
Stacker 1	CD-R	CD-R
Stacker 2	DVD(+R/-R)	CD-R
Stacker 3	Output Stacker	CD-R
Drive to use	Use Both	
Number of write retries	3	3
Printer Name	EPSON PP-100PRN	EPSON PP-50PRN
Show error mark on discs	Select	Select

EPSON TD Bridge settings

To configure EPSON TD Bridge, see "Configure EPSON TD Bridge" and enter the settings described in the following table.

Item	Setting
Monitoring Folder	C:\EPSON\TDBridge\Orders
TD Bridge Startup Type	Automatic (when starting PC/service)
Default output device	pacsgear
Default output stacker in standard mode	Stacker 3 or Stacker 4
Label Type	CD/DVD Label