



# Media Writer Installation Guide

DOC-MW6.0-IN-EN-REVB

## Copyright

Information in this document is subject to change without notice. The software described in this document is furnished only under a separate license agreement and may be used or copied only according to the terms of such agreement. It is against the law to copy the software except as specifically allowed in the license agreement. This document or accompanying materials contains certain information which is confidential information of Hyland Software, Inc. and its affiliates, and which is subject to the confidentiality provisions agreed to by you.

All data, names, and formats used in this document's examples are fictitious unless noted otherwise. Complying with all applicable copyright laws is the responsibility of the user. Without limiting the rights under copyright law, no part of this document may be reproduced, stored in or introduced into a retrieval system, or transmitted in any form or by any means (electronic, mechanical, photocopying, recording, or otherwise), or for any purpose, without the express written permission of Hyland Software, Inc. or one of its affiliates.

Hyland®, Hyland Software®, Hyland Healthcare, and Hyland product names are registered and/or unregistered trademarks of Hyland Software, Inc. and its affiliates in the United States and other countries. All other trademarks, service marks, trade names and products of other companies are the property of their respective owners.

© 2021 Hyland Software, Inc. and its affiliates. All rights reserved.

## Regulatory and compliance

<b>Australian Sponsor:</b> Emergo Australia Level 20 Tower II Darling Park 201 Sussex Street Sydney, NSW 2000 Australia	Media Writer Software Anvisa n°: 80117580229 Responsável Técnico: Luiz Levy Cruz Martins CRF-SP: 42415 IMPORTADOR: Emergo Brazil Import Importação e Distribuição de Produtos Médicos Hospitalares Ltda. Avenida Francisco Matarazzo, 1.752, Salas 502/503, Água Branca, São Paulo-SP, CEP - 05001-200 CNPJ: 04.967.408/0001-98 e-mail: brazilvigilance@ul.com
<b>EU Representative:</b>  <b>EMERGO EUROPE</b> Prinsessegracht 20 2514 AP The Hague The Netherlands	



**Manufacturer**  
**Hyland LLC**  
4309 Hacienda Drive,  
Suite 500  
Pleasanton, CA 94588  
USA

**Support**  
Telephone: 1-844-535-1404  
Web: <https://community.hyland.com>

## General warnings and precautions



To ensure patient safety and adhere to HIPAA Privacy Rule involving patient health information, the Healthcare provider shall assign an unique Accession Number and Medical Record Number (MRN) to identify individual patient's health record affiliated with the patient to prevent erroneous medical data and duplicate identification with other patients.

# Symbols glossary



ISO - 15223-1 - Medical devices - Symbols to be used with medical device labels, labeling, and information to be supplied.

3082 - Manufacturer: Indicates the medical device manufacturer.



ISO - 15223-1 - Medical devices - Symbols to be used with medical device labels, labeling, and information to be supplied.

Authorized representative in European Community: Indicates the authorized representative in the European union. Symbol is accompanied by the name and address of the authorized representative adjacent to the symbol.



ISO - 15223-1 - Medical devices - Symbols to be used with medical device labels, labeling, and information to be supplied.

2497 - Date of Manufacture: Indicates the date when the medical device was manufactured.



ISO - 15223-1 - Medical devices - Symbols to be used with medical device labels, labeling, and information to be supplied.

2493 - Catalog number: Indicates the manufacturer's catalog number so the medical device can be identified.



ISO - 15223-1 - Medical devices - Symbols to be used with medical device labels, labeling, and information to be supplied.

2498 - Serial number: Indicates the manufacturer's serial number so that a specific medical device can be identified.



ISO - 15223-1 - Medical devices - Symbols to be used with medical device labels, labeling, and information to be supplied.

1641 - Consult instructions for use: Indicates the need for the user to consult the instructions for use.



ISO - 15223-1 - Medical devices - Symbols to be used with medical device labels, labeling, and information to be supplied.

0434A - Caution: Indicates the need for the user to consult the instructions for use for important information such as warnings and cautions.



ISO - 15223-1 - Medical devices - Symbols to be used with medical device labels, labeling, and information to be supplied.

2610 - Patient number: Indicates a unique number associated with an individual patient.



Medical Devices Directive 2017/745/EC

CE Marking: Indicates that a product complies with applicable European Union regulations.



Quantity: Indicates the number of units.



Directive 2012/19/EU on waste electrical and electronic equipment (WEEE).



ISO/DIS ISO 15223-1 - Medical devices - Symbols to be applied to be used with medical device labels, and information to be supplied.

Indicates the item is a medical device.

# Table of Contents

<b>Copyright .....</b>	<b>2</b>
<b>Regulatory and compliance .....</b>	<b>3</b>
<b>General warnings and precautions .....</b>	<b>3</b>
<b>Symbols glossary .....</b>	<b>4</b>
<b>What is Media Writer? .....</b>	<b>7</b>
<b>Download the Media Writer setup program .....</b>	<b>7</b>
<b>Install the Media Writer service .....</b>	<b>7</b>
<b>Install the Media Writer client .....</b>	<b>8</b>
<b>License Media Writer client .....</b>	<b>8</b>
<b>Log in for the first time .....</b>	<b>9</b>
<b>Install the Media Writer Web client .....</b>	<b>9</b>
<b>Launch the Media Writer Web client .....</b>	<b>9</b>

## What is Media Writer?

Media Writer is a program intended to enable physicians and clinicians to burn studies onto CDs, DVDs, or USB flash drives. This makes studies available to other physicians and patients, who can view their studies on a PC. With the Media Writer System CD/DVD burner, physicians can also design and print labels on the CDs and DVDs. Media Writer is not for diagnostic use.

## Download the Media Writer setup program

To download the Media Writer setup program to the host computer, perform one of the following actions.

- From a browser, go to the URL provided by your support representative. On the Download page, click the link. If prompted, specify a destination folder and then click OK.
- Insert the Media Writer installation disc into the CD-ROM drive of the host computer. In Windows, copy the installation folder from the CD-ROM to the host computer.

## Install the Media Writer service

To install the Media Writer service, complete the following steps.

1. Navigate to the *PACSGEAR Media Server for MediaWriter* folder, and then double-click *setup.exe*.
2. If the Windows UAC prompt appears, click **Yes**.
3. In the language selection dialog box, click a language in the list and then click **OK**.
4. In the **Welcome to the InstallShield Wizard for MediaWriter Server** dialog box, click **Next**.
5. In the **License Agreement** page, scroll to the bottom of the license agreement, and then click **Yes**.
6. In the **Choose Destination Location** dialog box, browse for a new location if necessary, and then click **Next**. The system displays the **Custom Setup** page.
7. If a PACSGEAR SQL instance is detected, click **Next** and then complete the following sub-steps.
  1. In the **Database server that you are installing to** list, select or browse for a database.
  2. Under **Connect using**, click **Windows authentication**.
  3. Optional. To use a local SQL server, click **SQL Server authentication**, and then type your SQL login ID and password.
  4. In the **Name of database catalog** box, browse for the catalog.
  5. Click **Next**.
8. If a PACSGEAR SQL instance is not detected and you want to install a new server, then complete the following sub-steps.
  1. In the **Microsoft SQL Server** window, confirm that the feature **Install SQL instance** is enabled.
  2. Click **OK** and then **Next**. The system begins installing the server.
  3. Follow the instructions to complete the installation.
9. If a PACSGEAR SQL instance is not detected, but you want to connect to an existing database instance on a separate computer on the network, complete the following sub-steps.
  1. Right-click **Install new SQL instance**, select **Disable** and then click **Next**.
  2. In the **Host** box, type the host or instance of the database server you want to use.
  3. In the **User name** and **Password** boxes, type server credentials.
  4. In the **Database** box, type the name of the database you want to use.
  5. Click **Continue**.

10. Click **Next**.
11. In the **Ready to Install the Program** page, click **Install**.  
InstallShield begins installing the server. If prompted, follow the on-screen instructions to install any missing requirements or prerequisites.
12. In the **InstallShield Wizard Complete** page, click **Yes**, I want to restart my computer now, and then click **Finish**.

## Install the Media Writer client

To install the Media Writer client, complete the following steps.

1. Navigate to the *MediaWriter Client* folder, and then double-click *setup.exe*.
2. If the Windows UAC prompt appears, click **Yes**.
3. In the **Select the language** dialog box, click a language in the list and then click **OK**.
4. In the **Welcome to the InstallShield Wizard for MediaWriter Client** dialog box, click **Next**.
5. In the **License Agreement** page, scroll to the bottom of the license agreement, and then click **Yes**.
6. In the **Choose Destination Location** dialog box, browse for a new location if necessary, and then click **Next**.
7. In the **Select Features** page, click **Next**.
8. In the **Ready to Install the Program** page, click **Install**.  
InstallShield begins installing Media Writer Client. If prompted, follow the on-screen instructions to install any missing requirements or prerequisites.
9. In the **InstallShield Wizard Complete** page, click **Finish**.

## License Media Writer client

To license Media Writer client, complete the following steps.

1. To launch the Media Writer client program, double-click the MediaWriter desktop icon.
2. In the **MediaWriter License** dialog box, in the **System ID** box, note the system ID.
3. To obtain a license key, send an email to [Key-pacsgear@Hyland.com](mailto:Key-pacsgear@Hyland.com). In the body of the email, type the following information:
  - Site name
  - City, State/Province
  - Country/Region
  - Product name
  - System ID
  - Department/Workstation
4. When you receive your license key, enter it in the **License key** box.
5. To install a demo, select the **Activate demo** check box. In the **Month**, **Date**, and **Year** boxes, type the expiration date that was included with your license key.
6. Click **OK** to display the **Log On** dialog box.



## Log in for the first time

After you complete the licensing process, the Log On dialog box appears. To log in, complete the following steps.

1. In the **User name** box, type `admin`.
2. Leave the **Password** box blank.
3. In the **Server** box, accept the default value of localhost or type the IP address of the PC on which you installed the Media Writer server.
4. In the **Port** box, accept the default value of 8000, type the port number of the PC on which you installed the Media Writer server, and then click **OK**.

## Install the Media Writer Web client

To install the Media Writer Web client, complete the following steps.

1. Download and install PACSgear Enterprise Server.
2. Refer to the *PACSgear Enterprise Server Installation Guide* and *PACSgear Enterprise Server User Help* for more information on setting up Media Writer Web.

## Launch the Media Writer Web client

To launch the Media Writer Web client, complete the following steps.

1. Launch a web browser.
2. In the **Address** box, type `https://<serverIP>/PGAPPS`, and then press **ENTER**.
3. On the home page, click **Media Writer** and then select **Submit Job**.
4. On the **Patient Information** page, select the **Output Destination** and **Lookup Source** from the **Settings** sidebar.
5. Enter an **MRN**, **Accession number** and **Patient name** and then click **Search**. The system displays a list of patients and studies.
6. Select one or more **Studies** and click **Burn**.
7. From the Burn options list, configure the following options.
  - Media type
  - Disc label design
  - Anonymization option
  - Encryption
  - Label notes
  - Number of copies
8. Click **Burn**.