



ModLink Release Notes

DOC-ML25-RN-EN-REVA

Documentation Notice

Information in this document is subject to change without notice. The software described in this document is furnished only under a separate license agreement and may only be used or copied according to the terms of such agreement. It is against the law to copy the software except as specifically allowed in the license agreement. This document or accompanying materials may contain certain information which is confidential information of Hyland Software, Inc. and its affiliates, and which may be subject to the confidentiality provisions agreed to by you.

Complying with all applicable copyright laws is the responsibility of the user. Without limiting the rights under copyright law, no part of this document may be reproduced, stored in or introduced into a retrieval system, or transmitted in any form or by any means (electronic, mechanical, photocopying, recording, or otherwise), or for any purpose, without the express written permission of Hyland Software, Inc. or one of its affiliates.

Hyland, HXP, OnBase, Alfresco, Nuxeo, and product names are registered and/or unregistered trademarks of Hyland Software, Inc. and its affiliates in the United States and other countries. All other trademarks, service marks, trade names and products of other companies are the property of their respective owners.

© 2025 Hyland Software, Inc. and its affiliates.

The information in this document may contain technology as defined by the Export Administration Regulations (EAR) and could be subject to the Export Control Laws of the U.S. Government including for the EAR and trade and economic sanctions maintained by the Office of Foreign Assets Control as well as the export controls laws of your entity's local jurisdiction. Transfer of such technology by any means to a foreign person, whether in the United States or abroad, could require export licensing or other approval from the U.S. Government and the export authority of your entity's jurisdiction. You are responsible for ensuring that you have any required approvals prior to export.

.

Regulatory and compliance



Manufacturer
HylandSoftware, Inc.
12919 Earhart Ave
Auburn, CA 95602 USA

Support

Telephone: 1-844-535-1404

Web: <https://community.hyland.com>

General warnings and precautions



To ensure patient safety and adhere to HIPAA Privacy Rule involving patient health information, the Healthcare provider shall assign an unique Accession Number and Medical Record Number (MRN) to identify individual patient's health record affiliated with the patient to prevent erroneous medical data and duplicate identification with other patients.

Symbols glossary



ISO - 15223-1 - Medical devices - Symbols to be used with medical device labels, labeling, and information to be supplied.

3082 - Manufacturer: Indicates the medical device manufacturer.



ISO - 15223-1 - Medical devices - Symbols to be used with medical device labels, labeling, and information to be supplied.

2497 - Date of Manufacture: Indicates the date when the medical device was manufactured.



ISO - 15223-1 - Medical devices - Symbols to be used with medical device labels, labeling, and information to be supplied.

2493 - Catalog number: Indicates the manufacturer's catalog number so the medical device can be identified.



ISO - 15223-1 - Medical devices - Symbols to be used with medical device labels, labeling, and information to be supplied.

2498 - Serial number: Indicates the manufacturer's serial number so that a specific medical device can be identified.



ISO - 15223-1 - Medical devices - Symbols to be used with medical device labels, labeling, and information to be supplied.

1641 - Consult instructions for use: Indicates the need for the user to consult the instructions for use.



ISO - 15223-1 - Medical devices - Symbols to be used with medical device labels, labeling, and information to be supplied.

0434A - Caution: Indicates the need for the user to consult the instructions for use for important information such as warnings and cautions.



ISO - 15223-1 - Medical devices - Symbols to be used with medical device labels, labeling, and information to be supplied.

2610 - Patient number: Indicates a unique number associated with an individual patient.



Quantity: Indicates the number of units.



Directive 2012/19/EU on waste electrical and electronic equipment (WEEE).

Table of Contents

- Documentation Notice 2**
- Regulatory and compliance 3**
- General warnings and precautions 3**
- Symbols glossary 4**
- Overview 6**
- ModLink 25.1 (February 2025) 6**
 - Features 6
 - Enhancements 6

Overview

ModLink is a program that automatically populates clinical worksheets and forms for dictation systems with values from modalities, saving manual dictation time. You use the ModLink client program to configure each of the processes listed below. ModLink runs these processes in the background as a Windows service according to your configuration.

- Receive messages containing measurements from modalities on specific ports, and place those messages into meaningful matching groups.
- Assign templates to matching groups.
- Map values in matching groups to template fields that correspond to custom fields in a third-party dictation system.
- Forward the mapped data to the dictation system.

ModLink 25.1 (February 2025)

Note: This release only upgrades from ModLink 3.0.3. For assistance in upgrading and migrating your ModLink installation, contact Hyland technical support.

Features

This release provides the following features.

- Updated ModLink to a web-based user interface.
- Added support for multiple concurrent users.
- Added support for user access levels.
- Added support for domain-based user access levels.
- Added additional modifiers for use in translations.
- Removed the requirement for an MML file.

Enhancements

This release provides the following enhancements.

- Resolved an issue with deleting ports.
- Resolved multiple issues with data mapping.
- Resolved an issue with receiving HOLOGIC DEXA reports.