

PACS Scan Film for Mammography System Quick Start Guide

Step 1: Install

1. Unpack the PC from the box. Plug the PC power cord into the PC and a grounded outlet.
2. Unpack the monitor from the box. Plug the power cord into the monitor and a grounded outlet. Plug the monitor cable into the monitor and the PC.

Step 2: Document Scanner (Optional)

1. Unpack the scanner from the box. Plug the power cord into the scanner and a grounded outlet.
2. Plug the USB cable into the scanner and the PC.

Step 3: VIDAR CAD PRO® Advantage Digitizer


1. Place the digitizer on the stand so that the vent on the bottom of the digitizer is over the cutout on the stand's rear bar. Secure the digitizer to the stand with the two thumbscrews.



2. Attach the film feeder to the digitizer base:

Note: The three screws and the connector on the bottom of the feeder mate with the holes and the connector on the digitizer base.

- Place the feeder on top of the digitizer base and slide the feeder connector into the digitizer connector.
- On the back of the feeder, tighten the three thumbscrews to secure the feeder to the digitizer. Hook the exit tray over the notched plastic edge just below the rollers so that the tray is parallel to the roller shaft.

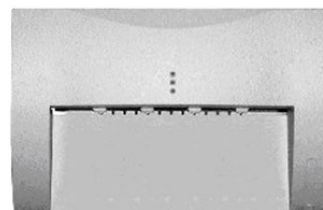

PACS Scan Film for Mammography System Quick Start Guide

Step 3: *continued...*

3. Hook the exit tray over the notched plastic edge just below the rollers so that the tray is parallel to the roller shaft.
4. Plug the power cable into the VIDAR digitizer and a grounded outlet.
5. Plug the USB cable into the VIDAR digitizer and the USB connector on the back of the PC.



Step 4: Power Up

1. Power up the VIDAR digitizer, monitor, and document scanner. Wait a few seconds for the VIDAR digitizer to start up, then power up the PC.
2. The PC automatically logs on with the "PACSGEAR" user account.
3. Change the system network IP address and user accounts if needed.

Step 5: Configure

1. On the desktop, double-click the PACS Scan shortcut. To add a user account, on the Settings menu, select Users. You can create new user accounts locally or use LDAP/Active Directory.
2. On the Settings menu, select DICOM.
3. In the DICOM Settings dialog box, type an AE title and add one or more DICOM store/print devices.
4. To enable patient lookup, select the Enable Patient Lookup check box. Configure an existing DICOM modality worklist or archive if applicable. Click OK.
5. You are now ready to use PACS Scan Film for Mammography!



Manufacturer
Hyland Software, Inc.
12919 Earhart Ave.
Auburn, CA 95602 USA

Support
Telephone: 1-844-535-1404
Web: <https://community.hyland.com>