



PACS Scan

Philips IntelliSpace Integration Guide

ENG-PS-IntelliSpace-IG-REVB

Date: November 2023

## Documentation Notice

Information in this document is subject to change without notice. The software described in this document is furnished only under a separate license agreement and may only be used or copied according to the terms of such agreement. It is against the law to copy the software except as specifically allowed in the license agreement. This document or accompanying materials may contain certain information which is confidential information of Hyland Software, Inc. and its affiliates, and which may be subject to the confidentiality provisions agreed to by you.

Complying with all applicable copyright laws is the responsibility of the user. Without limiting the rights under copyright law, no part of this document may be reproduced, stored in or introduced into a retrieval system, or transmitted in any form or by any means (electronic, mechanical, photocopying, recording, or otherwise), or for any purpose, without the express written permission of Hyland Software, Inc. or one of its affiliates.

Hyland, HXP, OnBase, Alfresco, Nuxeo, and product names are registered and/or unregistered trademarks of Hyland Software, Inc. and its affiliates in the United States and other countries. All other trademarks, service marks, trade names and products of other companies are the property of their respective owners.

© 2023 Hyland Software, Inc. and its affiliates.

The information in this document may contain technology as defined by the Export Administration Regulations (EAR) and could be subject to the Export Control Laws of the U.S. Government including for the EAR and trade and economic sanctions maintained by the Office of Foreign Assets Control as well as the export controls laws of your entity's local jurisdiction. Transfer of such technology by any means to a foreign person, whether in the United States or abroad, could require export licensing or other approval from the U.S. Government and the export authority of your entity's jurisdiction. You are responsible for ensuring that you have any required approvals prior to export.

## Revision History

| Date        | Revision | Authors        | Description   |
|-------------|----------|----------------|---|
| 23-OCT-2023 | B        | Tammy Matthews | Rebranded for Hyland Software, Inc.<br>Changed iSite references to IntelliSpace.<br>Minor formatting updates. |
|             |          |                |   |
|             |          |                |   |

# Table of Contents

**Documentation Notice..... 2**

**Revision History ..... 3**

**Scope ..... 5**

**Install a plugin ..... 5**

**Shortcut Commands ..... 6**

**Run PACS Scan Commands ..... 6**

    From an image..... 6

*Example* ..... 6

## Scope

This integration guide applies to PACS Scan and PACS Scan Film.

## Install a plugin

By installing one or more PACSgear plugins, you can launch PACSgear programs automatically from within IntelliSpace. To install a plugin, do the following.

1. In Windows Explorer, navigate to one of the following folders depending on your version of IntelliSpace.

*C:\Program Files\Pacsgear\PacsScan\integrations\iSite\iSite\_Radiology C:\Program Files\Pacsgear\PacsScan\integrations\iSite\iSite\_Enterprise*

The following table describes the plugins included in the folder, where “xxx” is “Enterprise” or “Radiology.”

| Plugin               | PACS Scan Function            |
|----------------------|-------------------------------|
| xxx_Plugin.htm       | Launch program with no action |
| xxxForm_Plugin.htm   | Launch electronic form        |
| xxxPaper_Plugin.htm  | Auto-scan paper               |
| xxxFilm_Plugin.htm   | Digitize film                 |
| xxxImport_Plugin.htm | Import images                 |

2. Write down the full path to a plugin, including the file name.

Example:

*C:\Program Files\PacsGear\PacsSCAN\integrations\iSite\iSite\_Radiology\Radiology\_Plugin.htm*

3. Launch **IntelliSpace** and open the **Preferences** dialog box.
4. Under **Machine Preferences**, click **Plug Ins**.
5. Click **Add**.
6. In the **Name** box, type a name for the plugin.
7. In the **URL** box, type the path to the plugin.
8. Select the **Enable in IntelliSpace Radiology/Enterprise** check box, and clear the **Disable API** check box.
9. Click **OK**.
10. Restart **IntelliSpace**.

## Shortcut Commands

The following commands appear in the shortcut menu for the patient list or viewer.

| Plugin               | PACS Scan Function |
|----------------------|--------------------|
| xxx_Plugin.htm       | Add Form           |
| xxxPaper_Plugin.htm  | Scan Paper         |
| xxxFilm_Plugin.htm   | Scan Film          |
| xxxImport_Plugin.htm | Import Image       |
| xxx_Plugin.htm       | Add Form           |

## Run PACS Scan Commands

### From an image

To run a PACS Scan command from an image, complete the following steps.

1. In **IntelliSpace**, open a study.
2. Right-click an image and click a command. PACS Scan launches and performs the command.

### Example

Click **Scan Paper**. PACS Scan launches and begins scanning documents

