

# PACS Scan Mobile

## Installation and Setup Guide

Version 1.1.0

---

Written by: Product Knowledge, R&D  
Date: October 2015  
LX-DOC-PSM-1.1.0-IN-EN-REVA



Lexmark™

---

© 2015 Lexmark International Technology, S.A. All rights reserved.

Lexmark is a trademark of Lexmark International, Inc., or its subsidiaries, registered in the U.S. and/or other countries. All other trademarks are the property of their respective owners. No part of this publication may be reproduced, stored, or transmitted in any form without the prior written permission of Lexmark.

## Table of Contents

<b>What is PACS Scan Mobile?</b> .....	<b>4</b>
<b>Download the PACS Scan Server setup program</b> .....	<b>4</b>
<b>Install PACS Scan Server</b> .....	<b>4</b>
<b>License PACS Scan Mobile</b> .....	<b>5</b>
<b>Change the administrator password</b> .....	<b>5</b>
<b>Install PACS Scan Mobile app and provision your device</b> .....	<b>5</b>

## What is PACS Scan Mobile?

PACS Scan Mobile is an app for mobile devices that physicians and authorized staff can use to capture images and videos easily. You can use the built-in camera of the mobile device to capture images or videos of your patient and then attach them to patient studies. When you are finished capturing images and videos you can edit them before sending them to the selected source for storage, and optionally send an HL7 message to an HL7 device. You cannot save images and videos on your mobile device.

This guide outlines the installation procedures for the PACS Scan Mobile app, PACS Scan Server, and licensing.

## Download the PACS Scan Server setup program

To download the PACS Scan Server setup program to the server computer, perform one of the following actions.

- From a web browser, go to the URL provided by your Lexmark representative. On the **Download** page, click the link provided. If prompted, specify a destination folder and then click **OK**.
- Insert the PACS Scan Mobile installation disc into the CD-ROM drive of the server computer. In Windows, copy the installation folder from the CD-ROM to the server computer.

## Install PACS Scan Server

To install PACS Scan Server, complete the following steps.

1. In the installation destination folder, double-click the **setup.exe** file.
2. If the message, "Do you want to allow the following program..." appears, click **Yes**.
3. In the **Welcome to the InstallShield Wizard for PACS Scan Mobile** dialog box, click **Next**.
4. In the **License Agreement** page, click **I accept the terms in the license agreement**, scroll to the bottom of the license agreement, and then click **Next**.
5. In the Destination Folder page, verify the installation destination folder. To change the folder, click **Change**.
6. Click **Next**.
7. In the **Ready to Install the Program** page, click **Install**.

If the installer fails to detect an installation requirement or prerequisite on the server computer, install the requirement or prerequisite by following the on-screen instructions.

8. In the **InstallShield Wizard Completed** page, click **Finish**.

**Note** PACS Scan Server runs automatically as a Windows service.

## License PACS Scan Mobile

To license PACS Scan Mobile, complete the following steps.

You must license PACS Scan Mobile before you can begin using it.

1. From a web browser, go to the **PSM Console** at *http://localhost/Admin*.  
**Note** If not logging in from the server computer, substitute *localhost* with the IP address of the server computer.
2. The first time you go to **PSM Console**, the **License** page opens.
3. In the **System ID** box, note the system ID.
4. Obtain a license key by sending an email to *key@lexmark.com*. In the body of the email, type the following information:  
Hospital Name, City, State, Country/Region, Product Name, System ID, and Department/Workstation.
5. When you receive your license key, enter it in the **License key** box.
6. To install a demo, select the **Enable demo** check box and enter the expiration date in the **Expiration** box.
7. Click **Save**.

After licensing PACS Scan Mobile, the Log in page opens.

## Change the administrator password

Change the administrator password in PSM Console by completing the following steps.

1. In the **Log in** page, complete the following actions.
  - In the **User name** box, type *admin*.
  - In the **Password** box, type *pacsgear1*.
  - Click **Log In**.
2. In the left pane, click **Users**.
3. Under **Assign access permissions**, click **Local Users**.
4. In the list of users, click the *admin* user name.
5. In the **Edit User** dialog box, type a new password in the **Password** and **Confirm password** boxes.
6. Click **Save**.

## Install PACS Scan Mobile app and provision your device

To install the PACS Scan Mobile app and provision your mobile device, complete the following steps.

**Prerequisite.** You must install and license the PACS Scan Server before you can install the app.

1. In the iTunes store, download and install the PACS Scan Mobile app.
2. On your mobile device, in Safari, log in to **PSM Console**.
3. On the **Dashboard**, click the link under **Click here to provision your iOS device**.