

PACS Scan Mobile

Installation Guide

Version: 2.0.1

Written by: Product Knowledge, R&D

Date: September 2016

LX-DOC-PSM2.0.1-IN-EN-REVB

Australian Sponsor

Emergo Australia
Level 20 Tower II
Darling Park 201 Sussex Street
Sydney, NSW 2000 Australia



Lexmark Enterprise Software, LLC

4309 Hacienda Drive, Suite 500
Pleasanton, CA 94588 USA



EMERGO EUROPE

Molenstraat 15
2513 BH, The Hague
The Netherlands



© 2016 Lexmark. All rights reserved.

Lexmark is a trademark of Lexmark International Inc., registered in the U.S. and/or other countries. All other trademarks are the property of their respective owners. No part of this publication may be reproduced, stored, or transmitted in any form without the prior written permission of Lexmark.

Table of Contents

What is PACS Scan Mobile?	4
Download the PACS Scan Server setup program.....	4
<i>Install PACS Scan Server</i>	4
<i>License PACS Scan Mobile</i>	5
<i>Change the administrator password</i>	5
Install the PACS Scan Mobile app and provision your device	5
<i>Log in to PACS Scan Mobile for the first time</i>	6
<i>Log in to PACS Scan Mobile with Touch ID</i>	6

What is PACS Scan Mobile?

PACS Scan Mobile is an app for Apple mobile devices that physicians and authorized staff can use to capture images and videos easily. You can use the built-in camera of the mobile device to capture images or videos of your patient and then attach them to patient studies. When you are finished capturing images and videos you can edit them before sending them to the selected source for storage, and optionally send an HL7 message to an HL7 device. You cannot save images and videos on your mobile device.

This guide outlines the installation procedures for the PACS Scan Mobile app, PACS Scan Server, and licensing.

Download the PACS Scan Server setup program

To download the PACS Scan Server setup program to the server computer, perform one of the following actions.

- From a web browser, go to the URL provided by your Lexmark representative. On the **Download** page, click the link provided. If prompted, specify a destination folder and then click **OK**.
- Insert the PACS Scan Mobile installation disc into the CD-ROM drive of the server computer. In Windows, copy the installation folder from the CD-ROM to the server computer.

Install PACS Scan Server

To install PACS Scan Server, complete the following steps.

1. In the installation destination folder, double-click the **setup.exe** file.
2. If the message, "Do you want to allow the following program..." appears, click **Yes**.
3. In the **Welcome to the InstallShield Wizard for PACS Scan Mobile** dialog box, click **Next**.
4. In the **License Agreement** page, click **I accept the terms in the license agreement**, scroll to the bottom of the license agreement, and then click **Next**.
5. In the Destination Folder page, verify the installation destination folder. To change the folder, click **Change**.
6. Click **Next**.
7. In the **Ready to Install the Program** page, click **Install**. If the installer fails to detect an installation requirement or prerequisite on the server computer, install the requirement or prerequisite by following the on-screen instructions.
8. In the **InstallShield Wizard Completed** page, click **Finish**.

Note PACS Scan Server runs automatically as a Windows service.

License PACS Scan Mobile

To license PACS Scan Mobile, complete the following steps.

You must license PACS Scan Mobile before you can begin using it.

1. From a web browser, go to the **PSM Console** at *http://localhost/Admin*.
Note If not logging in from the server computer, substitute *localhost* with the IP address of the server computer.
2. The first time you go to **PSM Console**, the **License** page opens.
3. In the **System ID** box, note the system ID.
4. Obtain a license key by sending an email to *key@lexmark.com*. In the body of the email, type the following information: Hospital Name, City, State, Country/Region, Product Name, System ID, and Department/Workstation.
5. When you receive your license key, enter it in the **License key** box.
6. To install a demo, select the **Enable demo** check box and enter the expiration date in the **Expiration** box.
7. Click **Save**.

After licensing PACS Scan Mobile, the Log in page opens.

Change the administrator password

Change the administrator password in PSM Console by completing the following steps.

1. In the **Log in** page, complete the following substeps.
 1. In the **User name** box, type *admin*.
 2. In the **Password** box, type *pacsgear1*.
 3. Click **Log In**.
2. On the menu bar, on the **Settings** menu, click **Users**.
3. Under **Restrict access permissions to**, click **Local Users**.
4. In the list of users, click the *admin* user name.
5. In the **Add/Edit User** dialog box, type a new password in the **Password** and **Confirm password** boxes.
6. Select the **Administrator** check box, and then click **Save**.

Install the PACS Scan Mobile app and provision your device

To install the PACS Scan Mobile app and provision your mobile device, complete the following steps.

Prerequisite. You must install and license the PACS Scan Server before you can install the app.

1. In the iTunes store, install, and open the PACS Scan Mobile app.
2. On a PC, log in to PSM Console.
3. On the PACS Scan Mobile app, click the QR code icon.
Step result. The camera of the mobile device turns on.
4. Point the camera at the QR code on the PC screen until the Settings Updated message appears.

Log in to PACS Scan Mobile for the first time

To log in to PACS Scan Mobile for the first time, complete the following steps.

1. In the log in screen, type your user name and password and click Log In.
2. Optional. To save your thumbprint, click Yes, and then press your thumb on the thumbprint reader.

Note This step is only available if you registered your thumbprint when you set up your mobile device.

Log in to PACS Scan Mobile with Touch ID

If you saved your thumbprint the first time you logged in, you can log in again using Touch ID. To log in using Touch ID, complete the following steps.

Prerequisite. Your mobile device must include a Touch ID function.

1. On the log in screen, tap the thumbprint icon.
2. Press your thumb on the thumbprint reader.